Roosin®


No.8 Yuandong Road,Kouan Town Gaogang,
Taizhou City,J iangsu Province,P.R.CHINA 225321
- 0523-86908085
0523- 86908351
11 info@roosin .com
0 http:// ww w.roosin . com

HYDROCOLLOID WOUND DRESSING

PRODUCT DESCRIPTION

Product is a kind of disposable sterile product, it is mainly composed of medical hot melt glue, sodium carboxymethyl cellulose (CMC), and a good air permeability medical PU. Hydrocolloid dressings are soft, light, smooth, firm, not irritant to the skin, etc, is one of the best products to absorb the wound exudates.

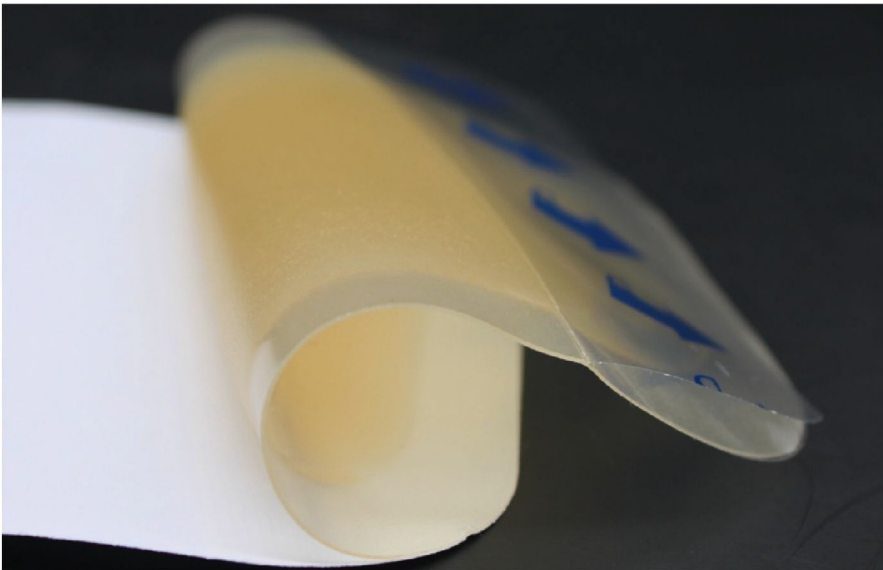STRUCTURE PERFORMANCE

1) Product composition and structure

The main raw material of the product is medical hot melt glue, sodium carboxymethyl cellulose (CMC). Hydrocolloid dressing is mainly composed of three layers, the bottom film (PU film/foam), the hydrocolloid layer and the release film:

2) Product performance description

- Liquid absorption
- Promote wound healing
- Self-adhesion

After absorbing the exudates, the hydrophilic particles (CMC) in the dressing can form semi-solid materials similar to gels, adhere to the surface of the wound, and provide and maintain a moist environment to wound healing. Meanwhile, the contact layer will form a gel when absorb the exudates, it is easy to remove and does not stick to the wound. Hydrocolloid dressing can absorb and lock the exudates into the dressing, and have the debridement function. It is sticky, and can form occlusive surface. The occlusive healing environment can promote the proliferation of micro vessels and the formation of granulation tissue. It is beneficial to scavenging necrotic tissue of macrophages to accelerate wound healing.


FUNDAMENTAL PRINCIPLE

- Wash the wound with normal saline.
- After the skin around the wound is dry, select the suitable size of the product (When necessary to cut the shape of the products), and the product should cover the wound completely.
- Fixed with secondary dressing.

METHOD OF USE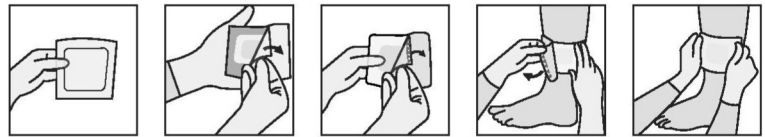

Apply to surgery or injury to cover, keep the wound moist, and absorb the exudates, contact with skin epidermis.

SCOPE OF APPLICATION

- Be careful with the infection of the wound.
- Patients who are known to be allergic to hydrocolloids are forbidden, cautiously use of allergic constitution.
- It is not recommended for dry wound.

CONTRAINDICATIONS

PRECAUTIONS

- This product is a disposable sterile product. It is strictly prohibited to reuse the product. Packaging damage is disabled.
- Please use it within the validity period. Read the instructions carefully before use.
- Before use of this product for refractory wound, should make a scientific analysis of wound types and causes and formulate corresponding treatment plans. Only this product can not be used as a refractory wound treatment scheme.

No corrosive gas, cool, wet ventilated, clean room, room temperature preservation.

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